

Cyber Energy Hydrogen Machine

user's manual

Connect the appliance to the mains. Switch on the appliance by pressing the release button. Pour a pack of potassium hydroxide into the main tank of the appliance. on top of pure water or reserve osmosis water. Add water to the front until the red light comes on. Stop the addition of water because the light is on.

Put the filter tank in pure water or reserve osmosis water. Fill until the front red light comes on. Close the tanks' lids.

The timer is set to 30 minutes. You can make changes from the timer if you want it to work differently.

Set the power setting to 10 amps. Do not exceed 15 amperes during operation. Use of more than 15 amperes may cause damage to the device

Fill the humidifier to the maximum level with clean water. If you want to be more refreshing to squeeze a little lemon into the water.

Attach the cannula to the humidifier. insert the other end into your nose. You can use 30-60 minutes cures 3 times a day. It can be used in more use if you want.

Maintenance of the device:

Drain the water in the main tank and filter tank after using for approximately 100 hours. Pour 800ml hydrochloric acid into the main tank. Leave for 5 minutes. then empty it. Fill and empty 3 times clean water. Then pour a pack of potassium hydroxide into the main tank and add water until the green light is on. Fill the filter tank with clean water until the red light comes on.

You can contact us through our e-mail address or whatsapp number to buy our products with payoneer at 15% discount price.

e-mail: berkanaktik@hotmail.com

whatsapp: +90 537 934 70 60