

S2 Cyber Energy Hydrogen Inhalation Device

User Manual

Device Setup

- 1. Power on the device:**

Plug the device into the socket and press the ON/OFF button on the front panel to start.
 - 2. Main tank preparation:**
 - Add **20 grams(1 tablespoon) of electrolyte chemical (potassium hydroxide)**
 - Then add purified water, reverse osmosis water, or bottled water.
 - Fill until the water level light turns on, then stop adding water.
 - The water should only cover the sensor. Do not overfill. Leave about **9 cm of empty space from the tank opening.**
 - 3. Fill the filter tank:**
 - Add only clean water into the filter tank.
 - Fill until the filter water level light turns on. Do not overfill. Leave about **9 cm of empty space from the tank opening.**
 - Close the tank lids tightly.
 - 4. Humidifier setup:**
 - Attach the metal holder of the humidifier cup to the screws on the left side of the device.
 - Fill the cup up to **3/4 with water.**
 - Connect the tube to the gas outlet port on the front panel of the device.
-

Usage

- When the water level lights on the front panel turn off, add only clean water to the corresponding tank until the light turns on again.
 - **Do not add chemicals** during normal use. Chemicals are only added if, after 15 minutes of operation, the device cannot reach the desired power. Then a small amount can be added gradually.
 - During operation, make sure the power does not exceed **350 watts**. If it does, reduce it using the power knob.
-

Timer and Power Settings

- **Default timer:** 45 minutes
- **To adjust the time:**
 - Press the "SET" button

- Use the UP/DOWN keys to set the duration
- **Maximum operating time:** 60 minutes
- After each session, allow the device to rest for at least **1 hour**

Power setting:

- For starting, set the power between **200–300 watts**.
 - Do not exceed **350 watts**. Otherwise, the device will enter **protection mode** and stop producing gas.
 - If the device shuts down:
 - Turn the power knob to minimum
 - Switch off the device
 - Wait 1–2 minutes, then restart
 - On the power display, **yellow numbers** show the real-time wattage. Adjust accordingly.
-

Periodic Maintenance

General Cleaning

- Every **150 hours**:
 - Empty the water from both tanks.
 - Fill each tank with clean water and empty it **3 times**.
 - Repeat the setup steps.
 - Every **300 hours (deep cleaning)**:
 - Pour **800 ml of 30% hydrochloric acid** into the main tank.
 - Wait 5 minutes, then empty.
 - Rinse the tank with clean water **3 times**.
 - Repeat the setup steps.
-

Filter and Dryer Maintenance

- ◆ **Activated Carbon Filter (Black container on the left):**
 - If you notice odor in the gas, replace the activated carbon.
 - It cannot be reused.
 - If the carbon has become wet, pour it into a fireproof container and dry it in a 100 degree oven for 1 hour.
 - Use activated carbon made from **coconut shell**.
 - Average lifetime: **6 months**
-

Additional Recommendations

- Change the water in the humidifier cup every **3 days**.

- The device display shows **total operating hours**. Follow this to perform maintenance on time.
- **Never ignite the produced gas**. This may cause an explosion inside the device and render it unusable. Such cases are **not covered by warranty**.

Usage Recommendations:

- Hydrogen remains in the blood for up to **2 hours**.
 - For serious health conditions: inhale for **45 minutes at 250–350 watts**, then take a **2-hour break** before the next session.
 - For healthy individuals: inhale **twice daily (morning & evening)** for **30–45 minutes** each session.
-


Warranty Conditions

- The device is covered by warranty for:
 - **2 years** for home use
 - **1 year** for commercial use
 - Exclusions:
 - Explosions caused by igniting the produced gas
 - Malfunctions due to running the device without water
-

Support

For any questions or support:

 Email: berkanaktik@hotmail.com

 WhatsApp: **+90 537 934 70 60**
